


SLING INSTRUCTION SHEET

TRI TURNER SLING


- 4) Attach the Tri Turner Sling Straps to the Lift hooks. Start Lifting the Sling. Make sure the Lift hooks are parallel to the patient's back Side. Cross or straighten legs before turning.

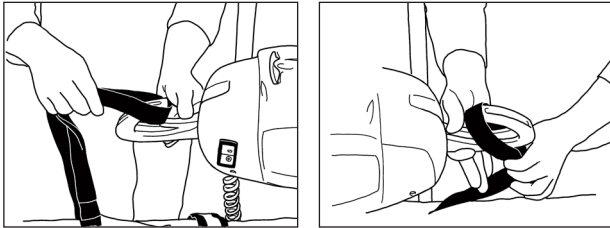

Note: Support the Patient while turning. Support Patient's back with pillow when giving bath.



- 6) After completing turning of the patient, lower the sling. Remove the Velcro loop and hook from the bed frame and fasten it onto itself before you pull it out from underneath the patient.

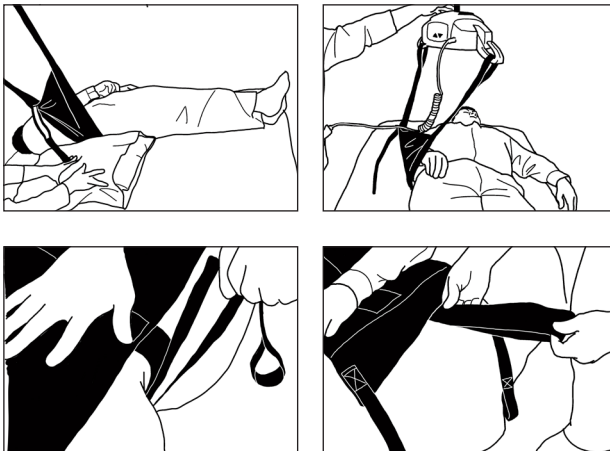

SPECIAL INSTRUCTIONS

This sling is appropriate for a client who has mild ability to participate in a lateral roll activity; may not appropriate for an unconscious client who has no active trunk control.

WASHING INSTRUCTIONS

The sling should be inserted into a washing/laundry bag prior to being placed into the washer. This is to prevent any unusual wear and tear of the sling by the agitator and/or other parts of the washing machine.



Note: The Tri Turner sling is made from polyester materials. This sling should last 3 to 5 years with normal use and care. Always follow general guidelines before use. Discard the sling if any tears or serious defects are found.

GENERAL GUIDELINES

Only people who have received proper training should attempt to use Handicare's Lift Systems. Improper use can lead to serious injury. Please read and adhere to the following guidelines:

1. Read all lift instructions before using.
2. Do not exceed safe weight capacity.
3. Visually inspect the sling prior to using it (check for cleanliness, fraying, cuts, or tears to straps and/or material).
4. A balanced fit is a safe fit: position the sling under and around the individual.
5. Do not use the lift systems for anything but their stated purpose.

All Handicare's loop-style slings are compatible with our complete range of carry bars.

To ensure anyone using Handicare's Lift Systems is properly trained, an on-going training program should be established. Contact your Handicare representative should you have further questions.

Handicare

485 Millway Avenue Unit 2, Concord ON L4K 3V4 Canada | 1.877.304.5438 | www.handicare.ca
10888 Metro Court, St. Louis, MO 63043 USA | Toll Free: 1.866.891.6502 | www.handicareusa.com